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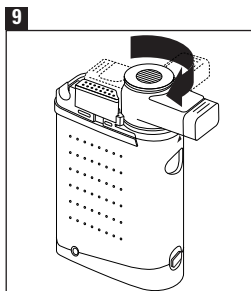
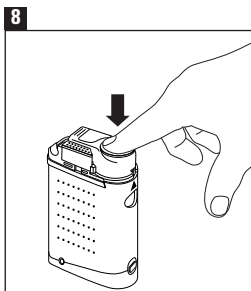
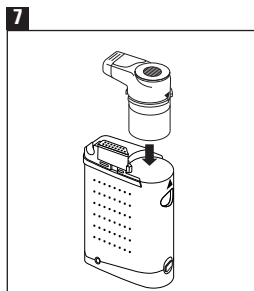
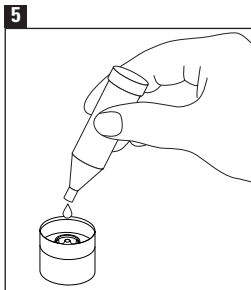
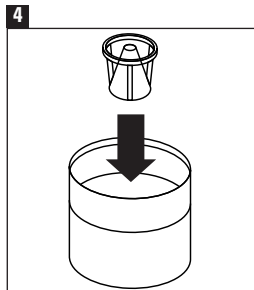
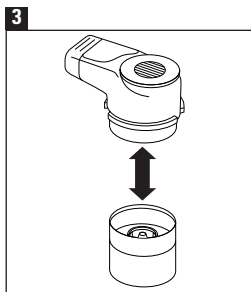
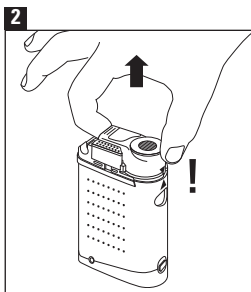
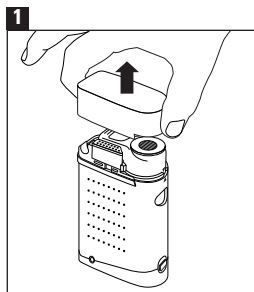
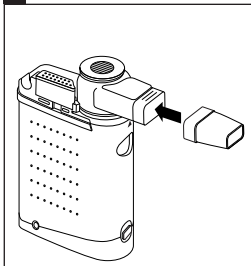
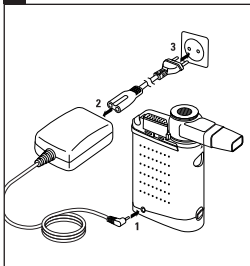
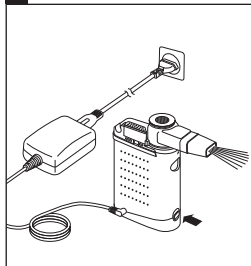
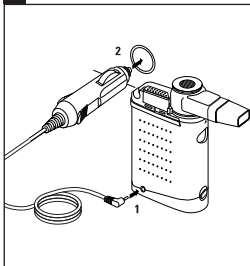
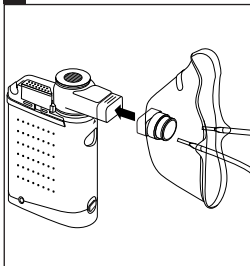
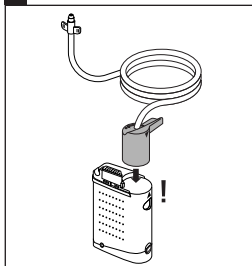
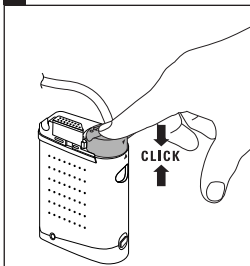
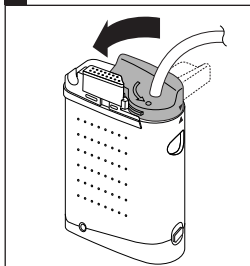
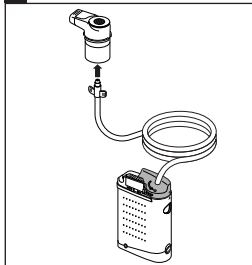
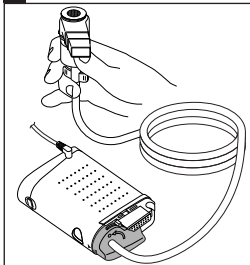
Table of contents

1. Introduction

2. Preparation and usage of the device

3. Using the hose extension

4. Operation using the 12V car cigarette lighter

5. Cleaning

6. Maintenance, care and service

7. Malfunctions and actions to take

8. Additional safety instructions

9. Guarantee

10. Technical specifications

1. Introduction

Your new Microlife nebuliser is a high-quality device for inhalation therapy for asthma, chronic bronchitis and other respiratory illnesses.

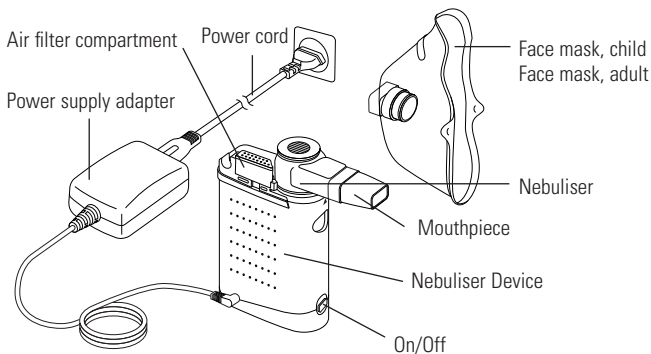
The device is capable of generating a medication mist of particles that are small enough to reach even the deepest regions of your lungs and be of maximal benefit.

Please only use the medication prescribed for you by your doctor and follow your doctor's instructions with regard to dosage, duration and frequency of the therapy. For safety reasons, the device and all its accessories should only be used by one person.

The operation and usage of the device is very straightforward. All common liquid medication can be used for inhalation therapy. Please do not use any ethereal oils, as these can react with the plastic materials used in inhalation devices.

Prior to using the device, please read this manual carefully and retain it in a safe place for future reference.

The figure shows the Microlife inhalation device:

2. Preparation and usage of the device

Prior to using the device for the first time, we recommend cleaning it as per the instructions in this manual.

Please refer to the foldout page with the figures on the correct usage of the device at the beginning of the manual.

1. Please remove the cover and then the nebuliser from the device and fill the nebuliser as shown in the figure with medication solution as per your doctor's instructions. Please ensure that you do not exceed the maximum level.

2. Pay attention to the position of the arrow when removing and fitting the nebuliser.
3. Please also ensure that the blue nozzle, as shown in the figures, is correctly positioned in the nebuliser container before you close the nebuliser. A spare nozzle is provided with the device.
4. The nebuliser must be pressed together until there is an audible «click» and firmly pressed into the device.
5. Rotate the nebuliser so that the front is facing you and fit the mouthpiece or face mask.
If possible use the mouthpiece as this gives a better drug delivery to the lungs. However, you may prefer to use the facemask and you will find two sizes are supplied to suit different size faces. The mask should completely enclose the mouth and nose areas.
6. Connect the cable from the power supply to the device as shown and connect the power supply to the electricity supply using the power cord (100 - 230V~ / 50 – 60 Hz).
7. Now switch on the device (I) and place the mouthpiece in the mouth or fit the mask. A green lamp illuminates above the switch during operation.
8. During the therapy, breathe in and out calmly. Sit in a relaxed position with the upper part of the body upright. Stop inhalation if you feel unwell.
9. After completion of the inhalation period recommended by your doctor, please switch off the device and unplug it from the electricity supply.
10. Empty the remaining medication from the nebuliser and clean the device as per the instructions in this manual.

3. Using the hose extension

As shown on the foldout page, you can use the hose extension between the device and nebuliser. Put the device down and conveniently remove the nebuliser from the device for inhalation.

1. To fit the hose extension, please proceed as shown in the figures.
2. Pay attention to the position of the arrow when removing and fitting the connecting piece.
3. Please also ensure that the connecting piece is firmly pressed onto the device and rotated until it is tight.

4. Operation using the 12V car cigarette lighter

Instead of power supply and power cord, you can also operate the device using the cable for the car cigarette lighter.

5. Cleaning

After inhalation the nebuliser should be dismantled as shown and placed in boiling water together with the mouthpiece for at least 10 minutes. Then place all parts on a clean paper towel and leave to dry.

- **The face masks and the hose extension must not be placed in hot water!**
- **The device and the power supply must not come into contact with water or heat!**

The face mask should be rinsed out with water after use and left to dry thoroughly. The device and the air hose should be wiped out once a day with a clean, moist cloth – **the power supply must not be plugged into the electricity supply during this process!**

For doctors' surgery, hospital:

The nebuliser chamber and mouthpiece can be placed in an autoclave at maximum 121 °C for 10 minutes. **The face masks, hose extension, power supply and device are not suitable for hot water or the autoclave!**

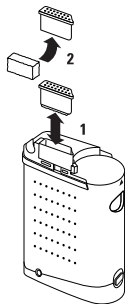
6. Maintenance, care and service

You can buy all necessary spare parts from your national Microlife customer service organisation. Please contact your dealer or pharmacist.

You will find detailed user information on our products and services at www.microlife.com

We recommend replacing the nebuliser, mouthpiece and masks after 3 months use.

The filter should be continually checked for cleanliness and replaced if dirty, or after 3 months use at the most. Spare filters are provided with the device. Open the air filter compartment on the device and replace the filter as shown.

7. Malfunctions and actions to take

Malfunction	Action
The device cannot be switched on.	<ul style="list-style-type: none"> • Check whether the power supply is connected correctly and plugged into the electricity supply correctly. The device has switched off automatically due to overheating. Wait until the device has cooled down and the red lamp above the switch is no longer illuminated.
The red lamp above the switch is illuminated.	<ul style="list-style-type: none"> • The device has overheated and cannot be used until the lamp is no longer illuminated.
The nebuliser only functions poorly or not at all.	<ol style="list-style-type: none"> 1. Check whether the nebuliser is firmly connected. 2. Make sure that the air hose is not kinked or blocked. 3. Check whether the nebuliser is correctly assembled. 4. Make sure that medication has been added.

The service team at Microlife will be pleased to provide assistance if you have any questions or problems!

8. Additional safety instructions

- If there is visible damage to the device, power supply or power cord, please stop using the device.
- The device is only allowed to be used for the purpose described in the instructions. The manufacturer is not liable for damages due to incorrect usage.
- Do not subject the device to extreme temperatures, moisture, dust or direct sunlight.
- Never try to repair the device yourself. Unauthorised opening of the device will render the guarantee void.
- Ensure that children do not use the device without supervision; some parts are so small they could be swallowed.
- If the device has received a hard knock (fallen on the floor), you find a malfunction or damage, or notice something unusual, you should have the device checked by your national Microlife customer service organisation.

9. Guarantee

This device is **guaranteed for 2 years** from the date of purchase. The guarantee covers the device, not the replaceable components like the nebuliser, masks, mouthpiece, hose extension and filter. The guarantee is void if the device is improperly used or changes made by third parties. The guarantee is only valid on presentation of the bill or a guarantee card completed by the dealer.

10. Technical specifications

Nebulisation rate:	0.35 ml/min. (3 ml inhalation takes less than 9 minutes)
Particle size:	2.0 µm (MMAD with 0.9% NaCl at API system) or 5.0 µm (MMD with 0.9% NaCl at Malvern system)
Compressor air flow:	5 l/min.
Acoustic noise level:	52 dBA at 1 m distance
Power source:	100 - 230 V~, 50 - 60 Hz
Power cord length:	1 m
Operating temperature:	10 - 40 °C
Operating humidity:	10 - 95% relative humidity maximum
Altitude:	up to 2000 m above sea level
Storage temperature:	-25 - +70 °C
Storage humidity:	10 - 95% relative humidity maximum
Weight:	310 g
Size:	38 mm x 78 mm x 119 mm
Reference to standards:	EN60601-1, EN60601-1-2, CE 0123

Subject to technical change without notice!

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